


CathGrip®

PEG Protect

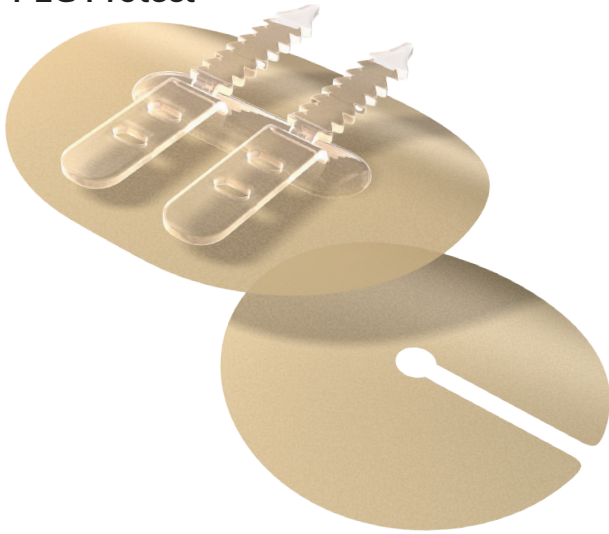

FOR INSTITUTIONAL USE ONLY

Application Tips

Make sure the patient's skin is COMPLETELY DRY before applying CathGrip®.

Cleansing wipes and Chlorhexidine can cause excess moisture underneath the skin that can reduce CathGrip wear time. Make certain that the application site for CathGrip is as dry and moisture free as possible, before applying.

Wear time. The first application for each patient will last shorter than the subsequent applications of the product. This is due to the hydrocolloid material that absorbs underlying moisture in the skin. CathGrip can be worn for up to 7 days.

Single-Use. Product is not reusable and could cause product malfunction or risk of infection if reused. Once removed, dispose of in a trash can.

Contraindications:

Absolute (When the product should not be used at all)

- Demonstrated sensitivity to hydrocolloid (rare)

Relative (When the product can be used after healing)

- Unhealed wound at application site (wait to heal)
- Active rash at application site (wait to heal)
- Active inflammation or infection

For use on intact skin.

If redness or irritation occurs, stop use and consult a healthcare practitioner.

Any serious incident while using this device should be reported to the manufacturer and competent authority.

NOT MADE WITH NATURAL RUBBER LATEX

bravida
medical

Manufactured by: Argentum Medical, LLC
2571 Kaneville Ct. Geneva, IL 60134
CustomerCare@BioDermInc.com
www.BravidaMedical.com



10°C (50°F) / 30°C (85°F)

©Argentum Medical, LLC DBA Bravida Medical. All rights reserved. BioDerm®, FreeDerm®, and CathGrip® are registered trademarks of BioDerm, Inc., DBA Bravida Medical. BioPlus+™ is a trademark of BioDerm, Inc., DBA Bravida Medical. Patents: <https://bioderminc.com/patents/>. Made in the USA with global components.

Removing CathGrip®

FreeDerm Instantly removes CathGrip with a gentle, quick drying, no washing, no-sting, skin friendly formula. Remove and replace CathGrip when the adhesive turns a white color, when it is loose, or when it is time to remove. We recommend using FreeDerm adhesive remover to remove CathGrip. You can also use warm water for removal by soaking CathGrip or wrapping a washcloth around the device until the hydrocolloid turns milky white and rolls gently from the skin.

To Remove with FreeDerm®:

Wipe or spray at the edge of the adhesive, lifting away from the skin as it releases. Continue until entire adhesive has been removed. Depending on the size of the adhesive, this may require more than one spray or wipe.



Sold separately in a box of 30 wipes (SKU: 90037). Also available as 1 oz. pump spray (SKU: 52200) and 3 oz. pump spray (52202)


Application Instructions

For Best Results
Follow the 5 P's

- Prep • Place • Peel
- Press • Pull


1

Preparation

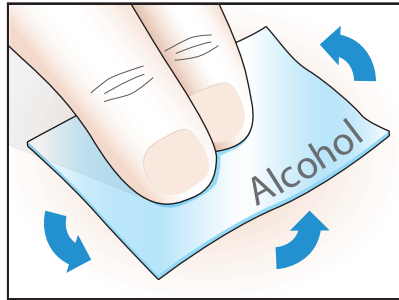


A Wipe & Dry

Clean both PEG and CathGrip application sites with alcohol wipe and/or non-moisturizing soap. Allow to dry.

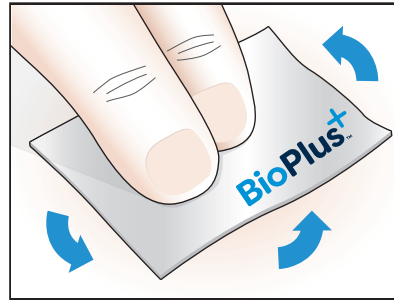

B Wipe & Dry


Wipe both PEG and CathGrip application sites entire application site with BioPlus+™. Allow 15+ seconds to dry.

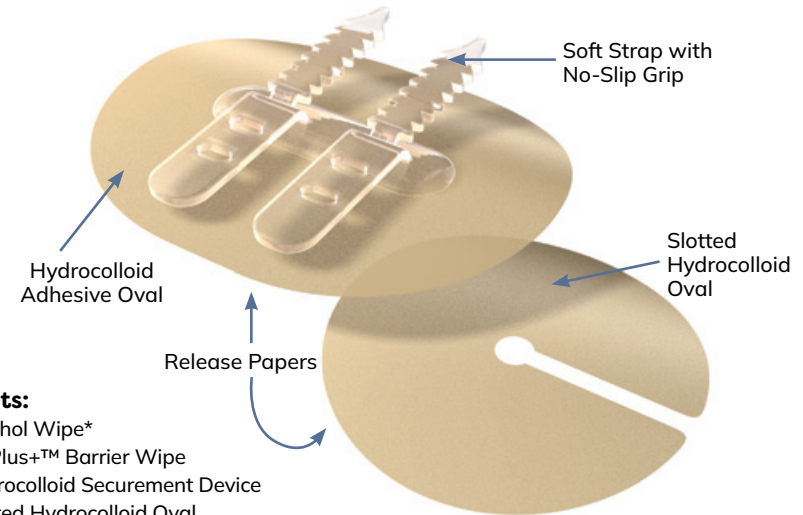

Contents:


- 1 Alcohol Wipe*
- 1 BioPlus+™ Barrier Wipe
- 1 Hydrocolloid Securement Device
- 1 Slotted Hydrocolloid Oval

*Not included in part numbers ending with NA

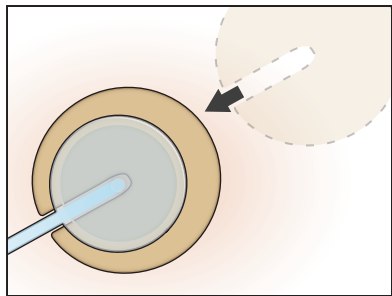


2 Place

Place the slotted hydrocolloid oval underneath the PEG bumper.

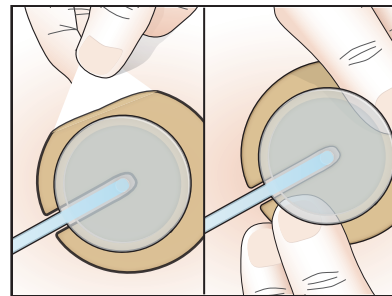


3 Peel & Press

Peel the release paper and press the slotted hydrocolloid oval to the skin for 15 seconds to seal.

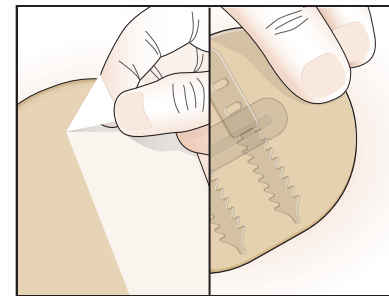

4 Peel & Press

Peel the release papers from the CathGrip with soft straps, place 4-6" from PEG and press for 15 seconds to seal.

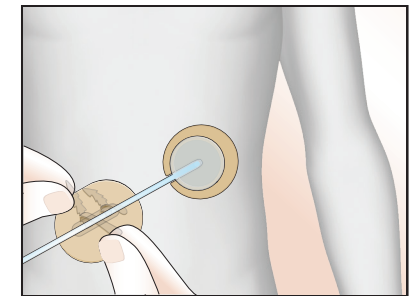

5 Pull & Secure

Place the tube on the soft, no-slip grips. Pull the straps tight to secure.

ATTENTION: Following application, leave a one inch space around the CathGrip® device when using any cleansing products, creams or lotions. Contact with the hydrocolloid will result in early dislodgement of the securement device.