


# Quick Reference Guide CathGrip® Application and Removal


## STEP 1: PREPARATION - CLEAN & PREP

### CathGrip® Universal Securement Low Profile Device

Made from “skin-friendly” hydrocolloid to protect and secure catheter in place for up to 7 days.

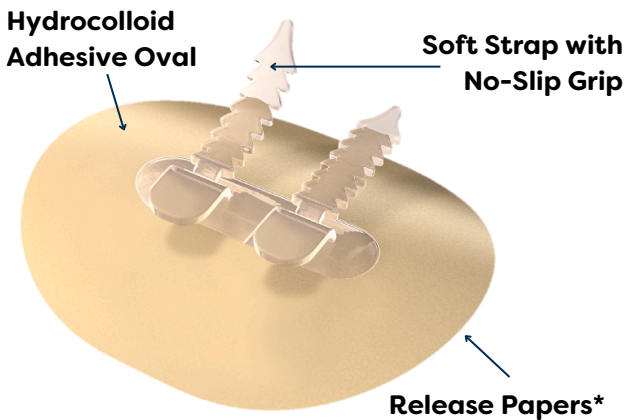

\*Release papers are located under the CathGrip® to protect the adhesive until ready to use.

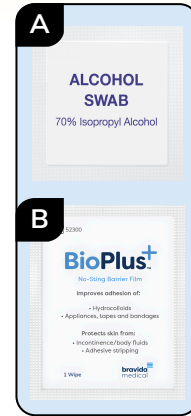


**CLEAN** skin by scrubbing with **ALCOHOL** wipe. Dry **COMPLETELY** for **15 SECONDS**.

**PREP** with **BioPlus+™** **BARRIER WIPE**. Dry **COMPLETELY** for **15 SECONDS**.


### PREP SECUREMENT AREA

- Trim hair (do not shave) prior to application if needed.
- Avoid belt line, locate area of smooth skin avoiding skin folds.
- Clean and prep entire application site area larger than CathGrip®.
- Allow both products to dry completely before applying CathGrip® to avoid early lifting.


## STEP 2: APPLICATION - PEEL & PRESS, THEN PLACE CATHETER

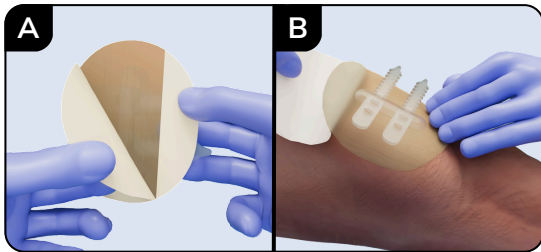


**PEEL** largest release paper first starting from the top corner downward at a 45-degree angle. Avoid touching the adhesive as much as possible.

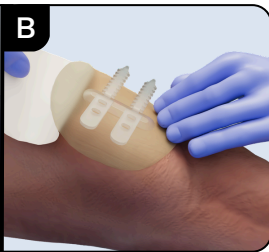


Apply CathGrip® to targeted securement area and **PRESS** into place. Gently remove the 2nd smaller release paper as shown.

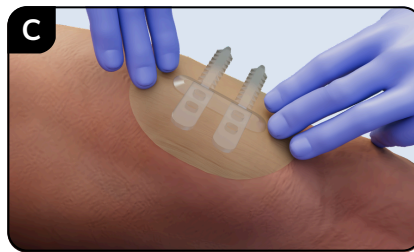

**PRESS** and **HOLD** the hydrocolloid oval in place at the center base and all edges for at least **15 SECONDS** to secure to skin.

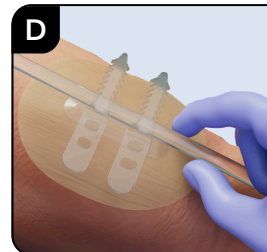

**PLACE** catheter on the soft, no-slip grips. **ALLOW** at least 1 inch of slack in catheter to avoid tension.

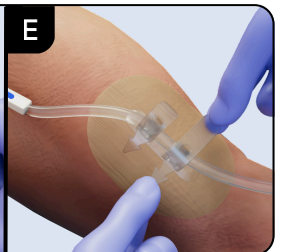

**PULL** soft straps tight around the catheter to secure. **LOCK** grips in place for secure hold by pulling both sides of the strap teeth fully through the opening.

**ATTENTION:** Following application, leave a one inch space around the CathGrip® device when using any cleansing products, creams or lotions. Contact with the hydrocolloid will result in early dislodgement of the CathGrip®.

**REMOVAL:** Remove and replace CathGrip® when the adhesive turns a white color, when it is loose, or when it is time to remove. We recommend using FreeDerm™ adhesive remover. Replace CathGrip® every 7 days or more frequently as needed. You can also use warm water for removal by soaking CathGrip® or wrapping a washcloth around the CathGrip® until the hydrocolloid turns milky white and rolls gently from the skin.

For important product and safety information including IFU, please visit, [www.bravidamedical.com](http://www.bravidamedical.com)