

Men's LibertyTM ACUTE



FOR INSTITUTIONAL USE ONLY

Contraindications:

Absolute (When the product should not be used at all)

- Demonstrated sensitivity to hydrocolloid (rare)
- Anatomic abnormality (severe acquired hypospadias, urethral fistula, penile shaft erosions)

Relative (When the product can be used after healing)

- Unhealed wound at application site (wait to heal)
- Active rash at application site (wait to heal)
- Active inflammation or infection of the glans, prepuce or urethra (wait to heal)
- Purulent or bloody urethral discharge (wait to heal)


Any serious incident while using the device should be reported to the manufacturer and competent authority.

We recommend you change the Men's Liberty Acute once a day. Do not wear more than 48 hours without applying a new unit. Using Men's Liberty with a diaper or other absorbent products may increase your risk of skin irritation or infection. If redness or irritation occurs, discontinue use and consult your healthcare professional.

NOT MADE
WITH
NATURAL
RUBBER
LATEX

bravida
medical

Manufactured by: Argentum Medical, LLC
2571 Kaneville Ct.
Geneva, IL 60134
CustomerCare@BioDermInc.com
www.BravidaMedical.com

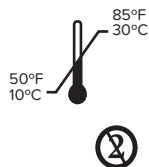

© Argentum Medical, LLC DBA Bravida Medical. All rights reserved. BioDerm® is a registered trademark of BioDerm, Inc DBA Bravida Medical. Men's Liberty Acute™, BioPlus+™ and KindKlamp™ are trademarks of BioDerm, Inc. DBA Bravida Medical. Made in the USA with global components. Patents: <https://bioderminc.com/patents/>

LC-IFU-BIO210268-01 B

Application Tips



Make sure the patients' anatomy is **COMPLETELY DRY** before applying Men's LibertyTM Acute.



Do **NOT** use any cleansing wipes, moisturizing creams, lotions or oils on the penis prior to application.



DO NOT use other products between Men's LibertyTM Acute applications (i.e. cleansing or chlorhexidine wipes). Excess skin moisture caused by other incontinence products will interfere with Men's Liberty Acute's seal, shortening wear time.



Do **Not** connect to suction.



Wear time. The first application for each patient will last shorter than the subsequent applications of the product. You can expect the first application to last 6-12 hours and subsequent applications to last 24-48 hrs. This is due to the hydrocolloid material that absorbs underlying moisture in the skin.

Product is not reusable and could cause product to malfunction or risk of infection if reused. Once removed, dispose of in a trash can.

Removing Men's Liberty Acute

FreeDerm® Instantly removes Men's Liberty Acute with a gentle, quick drying, no washing, no-sting, skin friendly formula. Remove and replace Men's Liberty Acute when the adhesive turns a white color, when it is loose, or when it has been soiled. We recommend using FreeDerm adhesive remover to remove Men's Liberty Acute. You can also use warm water for removal by soaking the petals or wrapping a washcloth around the device until the hydrocolloid turns milky white and rolls gently from the skin.


To Use:

Wipe or spray at the edge of the adhesive, lifting away from the skin as it releases. Continue until entire adhesive has been removed. Depending on the size of the adhesive, this may require more than one spray or wipe.


Sold separately in a box of 30 wipes (SKU: 90037). Also available as 1 oz. pump spray (SKU: 52200).



Application Instructions

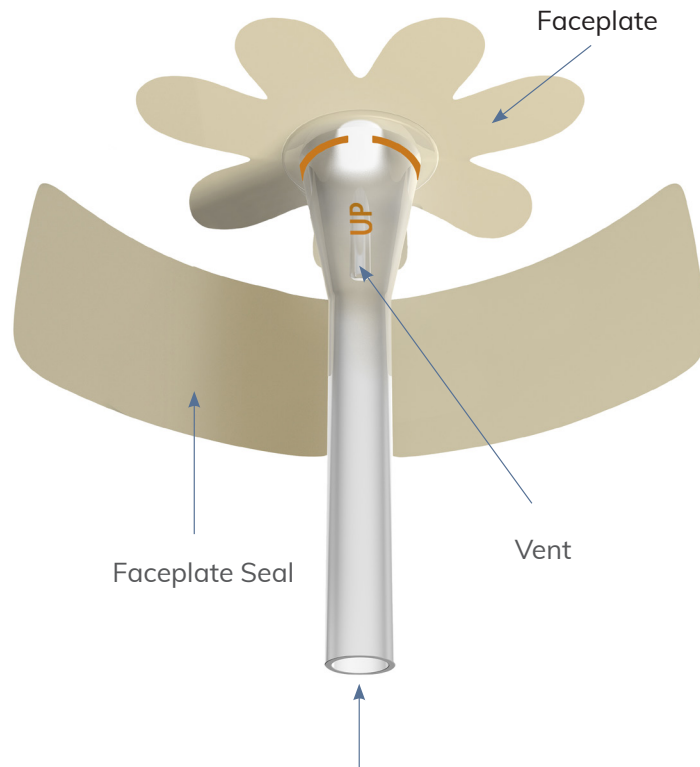

Connects to any leg or drain bag
One size fits most incontinence patients

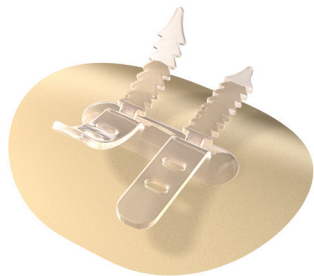

CathGrip®
Double Strap

CathGrip® Double Strap Large is the recommended device to anchor Men's Liberty Acute and is included in the ML with CathGrip set.

1 Wash & Dry

Always begin with a clean and dry penis. Keep foreskin back completely throughout application. To begin, wipe the head of the penis with the alcohol wipe avoiding the urinary meatus and allow to dry.

2 Pre-Application Prep

Wipe the head of the penis with BioPlus+™ wipe avoiding the urinary meatus. Allow 15-20 seconds to dry until the skin is no longer tacky.

3 Positioning & Application

Position Men's Liberty Acute with the vent facing up, as indicated by the printed "UP" on the vent and printed "UP" on the Faceplate release paper. Remove release papers around the petals and attach the faceplate to the tip of the penis with the center opening directly in line with the urinary opening. Smooth down petals.


4 Secure and Seal

Apply the faceplate seal around the head of the penis, flat and wrinkle free. Faceplate Seal should cover the Faceplate petals and the base of the housing tube up to the printed yellow lines on the tube. Be sure the adhesive material is pressed into all of the ridges and grooves in the skin. Press and hold for 10-15 seconds to secure the seal. After catheter application on uncircumcised patients, allow the foreskin to fall back to its natural position.

5 Connect to Leg or Drain Bag

Connect to Leg or Drain Bag and be sure to keep below the bladder or bed to ensure drainage. Do not twist or kink the tube. For best results, use CathGrip® Double Strap to secure bag and reduce tension.

Wipe & dry completely

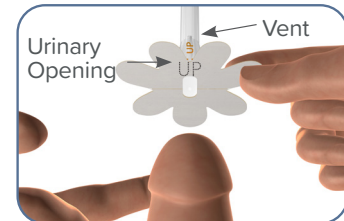

Center seal under both the tube and tip of penis



Press to smooth and remove wrinkles

