Anasept® Antimicrobial Skin & Wound Cleanser
Anasept® Antimicrobial Wound Irrigation Solution

Product Description:

Anasept® Antimicrobial Skin & Wound Cleanser and Irrigation Solution are clear, non-sterile, isotonic liquids that help in the mechanical removal of debris and foreign material from the application site. Dirt, debris, and foreign materials are mechanically removed by the action of the fluid moving across the wound bed or application site.

Anasept® Antimicrobial Skin & Wound Cleanser and Irrigation Solution delivers 0.057% broad spectrum antimicrobial sodium hypochlorite inhibiting the growth of bacteria such as *Acinetobacter baumannii*, *Clostridium difficile*, *Escherichia coli*, *Proteus mirabilis*, *Pseudomonas aeruginosa*, *Serratia marcescens*, *Staphylococcus aureus*, and Carbapenem resistant *E. Coli* (CRE), Methicillin resistant *Staphylococcus aureus* (MRSA), and Vancomycin resistant *Enterococcus faecalis* (VRE), that are commonly found in wound bed, as well as fungi such as *Aspergillus niger*, *Candida albicans* and *Candida auris*.

Available as:

<u>Code</u>	<u>Size</u>	<u>Code</u>	<u>Size</u>
4004C	4 oz. PET bottles with dispensing cap	4016C	15 oz. HDPE bottles with dispensing cap
4004SC	4 oz. PET bottles with finger sprayer	4008TC	8 oz. HDPE bottles with trigger sprayer
4008C	8 oz. PET bottles with dispensing cap	4012SC	12oz. HDPE bottles with trigger sprayer
4008SC	8 oz. PET bottles with finger sprayer	4160IC	16 oz. PET bottles with spikeable cap

Indications:

OTC Use: Anasept® Antimicrobial Skin & Wound Cleanser and Irrigation Solution are intended for OTC use for mechanical cleansing and removal of dirt, debris and foreign material from skin abrasions, lacerations, minor irritations, cuts, exit sites and intact skin.

Professional Use: Anasept® Antimicrobial Skin & Wound Cleanser and Irrigation Solution are intended for use under the supervision of a healthcare professional for cleansing of foreign materials including micro-organisms from wounds such as stage I-IV pressure ulcers, diabetic foot ulcers, post-surgical wounds, first and second degree burns, grafted and donor sites.

Directions for Use:

Anasept® Antimicrobial Skin & Wound Cleanser

Skin:

- 1) Spray the intended area or saturate gauze and apply to site.
- 2) Let air dry or maintain as a wet dressing.

Wounds:

- 1) Spray or saturate gauze pad and apply to wound surface.
- 2) Avoid pooling and cover with appropriate wound dressing.
- 3) Repeat as necessary.

Anasept® Antimicrobial Wound Irrigation Solution

Must be used with a vented spike adapter set

- 1) Remove dust cover from the cap.
- 2) Insert vented spike.
- 3) Pull up the sling and hang on the pole or NPWT device.

- 4) Use Anasept® Antimicrobial Wound Irrigation Solution via NPWT device (follow device manufacturer's installation instructions).
- 5) Do not rinse.
- 6) Repeat if necessary

For external use only. Must not be injected or applied to the eyes.

Store at Room Temperature up to 25°C (77°F). Do not freeze.

The product has a 24-month shelf-life when stored at recommended temperatures.

Anasept® skin & wound care products are environmentally safe. Post-application, products may be disposed of in a sink or sewer system.

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