Anasept® Antimicrobial Skin & Wound Gel

Product Description

Anasept® Antimicrobial Skin & Wound Gel is clear, amorphous, isotonic hydrogel that helps maintain moist wound environment that is conducive to healing, by either absorbing wound exudate or donating moisture while delivering 0.057% broad spectrum antimicrobial sodium hypochlorite.

Anasept® Antimicrobial Skin & Wound Gel inhibits the growth of microorganisms, such as *Acinetobacter baumannii*, *Clostridium difficile*, *Escherichia coli*, *Proteus mirabilis*, *Pseudomonas aeruginosa*, *Serratia marcescens*, *Staphylococcus aureus*, Carbapenem resistant *E. Coli* (CRE), Methicillin resistant *Staphylococcus aureus* (MRSA), and Vancomycin resistant *Enterococcus faecalis* (VRE), that are commonly found in wound bed, as well as fungi such as *Aspergillus niger*, *Candida albicans* and *Candida auris*.

Available as:

<u>Code</u>	<u>Size</u>
-------------	-------------

5003G	3.0 oz. tube with dispensing cap
-------	----------------------------------

5015G	1.5 oz. tube with dispensing cap
-------	----------------------------------

Indications:

OTC Use: Anasept Gel® is intended for management of skin abrasions, minor irritation, lacerations, cuts, exit sites and intact skin.

Professional Use: Anasept Gel® is intended to be used under the supervision of a healthcare professional in the management of wounds such as stage I-IV pressure ulcers, partial & full thickness wounds, diabetic foot & leg ulcers, post-surgical wounds, first & second degree burns, grafted and donor sites.

Directions for Use:

Skin:

- 1) Cleanse site with appropriate skin cleanser.
- 2) Apply thin layer of Anasept® Gel to intended site. Repeat as necessary.

Wound:

- 1) Cleanse and/or debride wound as necessary.
- 2) Apply generous amount of Anasept® Gel directly onto wound bed. (1/8" to 3/16" thick).
- 3) Cover the wound with sterile gauze or other appropriate wound dressing and secure in place. (Avoid wound dressings containing silver and other metals).
- 4) Maintain a moist wound environment between dressing changes.

EXTERNAL USE ONLY.

NOT FOR OPHTHALMIC USE.

Store at Room Temperature up to 25°C (77°F). Do not Freeze.

The product has a 24-month shelf-life when stored at recommended temperatures.

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