


CATHGRIP® TUBE SECUREMENT DEVICE

HYDROCOLLOID, MEDIUM, DOUBLE STRAP, DEVICE SIZE: 3 IN. X 3.5 IN.





COMPATIBLE TUBING: 6-42 FRENCH

INSTRUCTIONS FOR USE

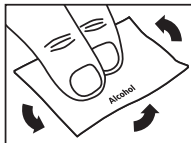

1. Prep

Clean entire application site with alcohol wipe and/or non-moisturizing soap and allow to dry. Then, wipe entire application site with BioPlus+™ barrier prep and allow 15+ seconds to dry.

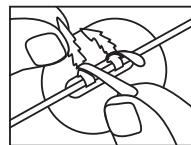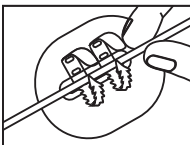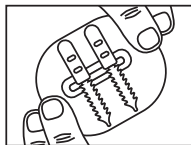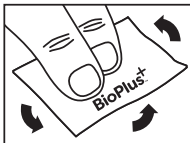

3. Press

Gently press the hydrocolloid oval to the skin for 15 seconds to seal.

4. Place

Place the tube on the soft, non-slip grips.

5. Pull

Pull the straps through the slot to secure the tube firmly.

CONTENTS:

- 1 Double Strap Oval
- 1 Alcohol Wipe
- 1 BioPlus+™ Barrier Wipe

For use on intact skin.

If redness or irritation occurs, stop use and consult a healthcare practitioner.

ATTENTION: Following application, do not apply cleansing products, creams or lotions within a one-inch area around the device. Contact with the hydrocolloid will result in early dislodgement of the securement device. Can be worn for up to 7 days.

NOT MADE WITH NATURAL RUBBER LATEX

1 EACH

©BioDerm, Inc., DBA Bravida Medical. All rights reserved. BioDerm® and CathGrip® are registered trademarks of BioDerm, Inc., DBA Bravida Medical. BioPlus+™ is a trademark of BioDerm, Inc., DBA Bravida Medical. Patents: <https://bioderminc.com/patents/>. Made in the USA with global components. Manufactured by Argentum Medical LLC, 2571 Kaneville Court, Geneva, IL, 60134, USA.

CathGrip®

